

Disassembly of Green X Unit Covers

This manual provides instructions on the proper disassembly of covers for Green X, Green X 18 and Green X 12 units.


I. Pre-requisite

I.I Tools Required

1.) Allen Wrench Set



2.) Phillips head screw driver w/ magnetic tip

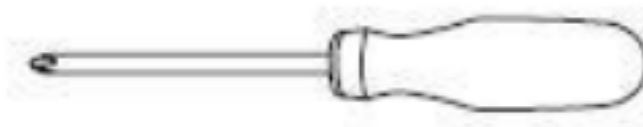


NOTICE


- The truss head screws on the cases of equipment are covered with rubber caps. Remove the rubber caps before unscrewing the truss head screws.
- In this manual, the procedures for removing the rubber caps are omitted for convenience.
- The following illustrations may look different from actual products depending on the options.

1. DISASSEMBLY OF THE VERTICAL UPPER COVER

1.1 Loosen all 6 screws from the top cover.

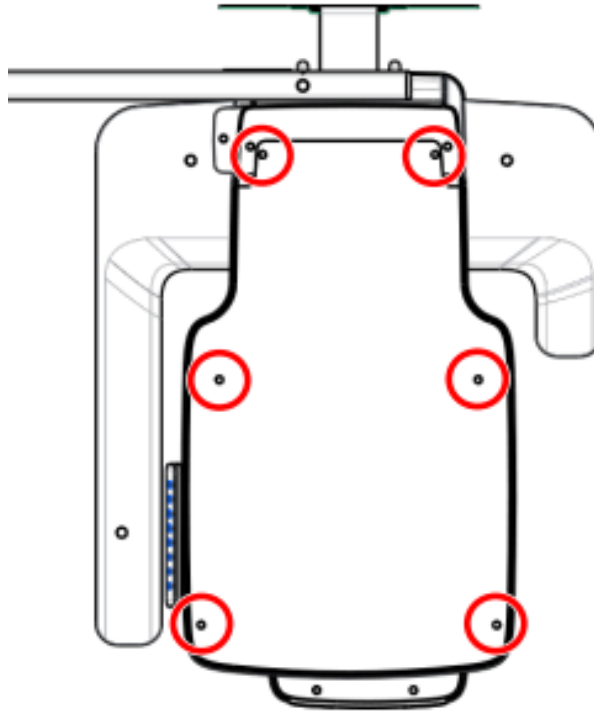


1.2 Lift the cover straight up to remove cover from the unit.

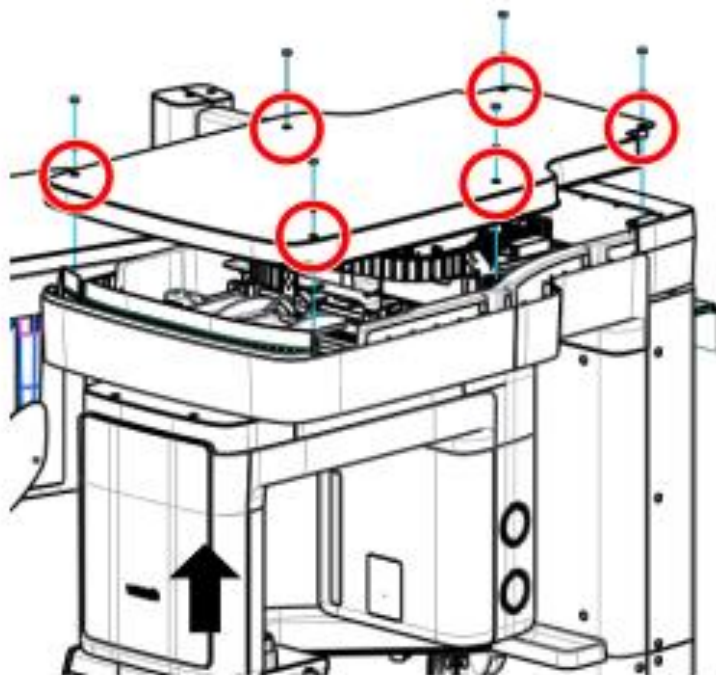


2. DISASSEMBLY OF THE ROTATOR COVERS


2.1 Loosen the four screws in the rotator top tube cover then remove.

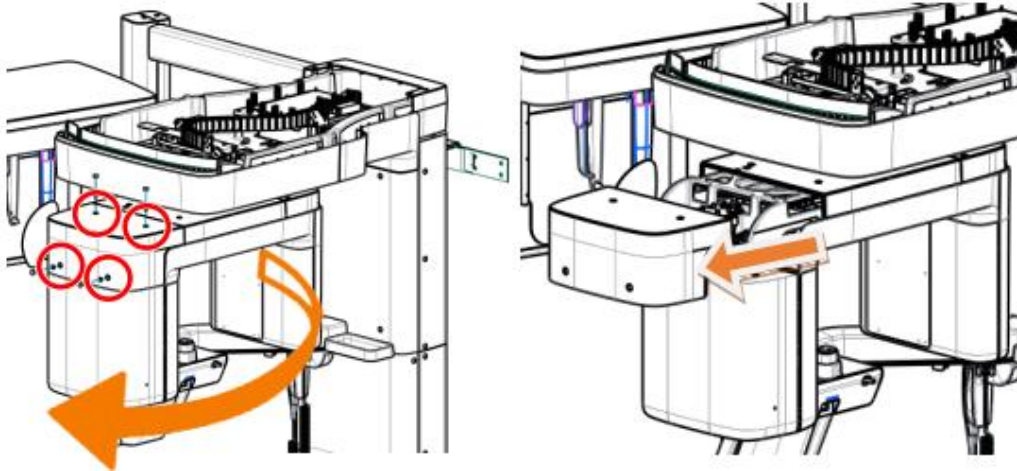

2.2 Loosen the two screws at the top and bottom of the rotator tube rear cover then remove.

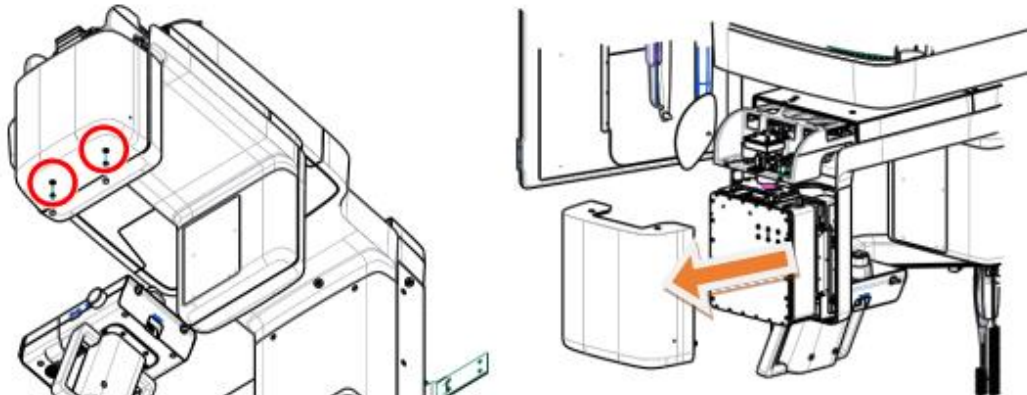


2.3 Loosen the six screws holding the rotator tube front cover to the bracket then remove.

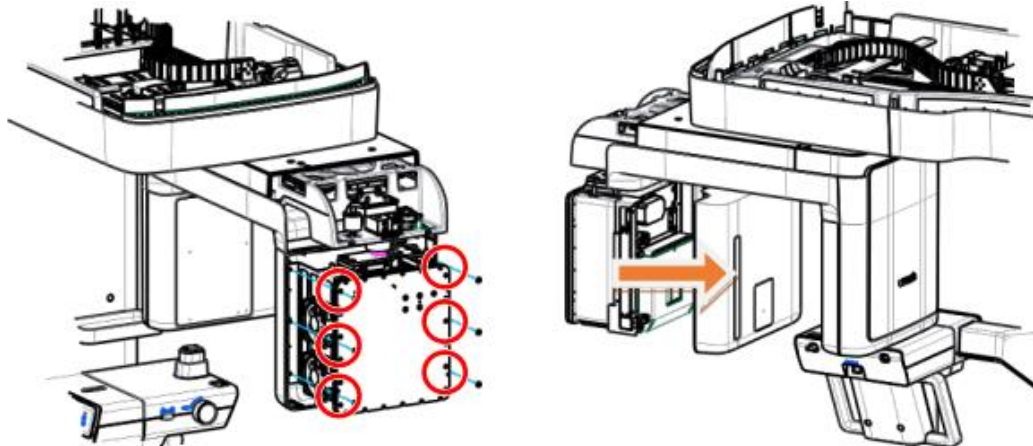


Disassembly of Green X Unit Covers


- 2.4 Loosen the four screws at the top of the rotator tube side upper cover to the rotator assembly. Some screws can be accessed through the opening in the vertical frame by repositioning the rotator until the screw is accessible in the opening. Once all screws have been removed, slide the cover out to remove.

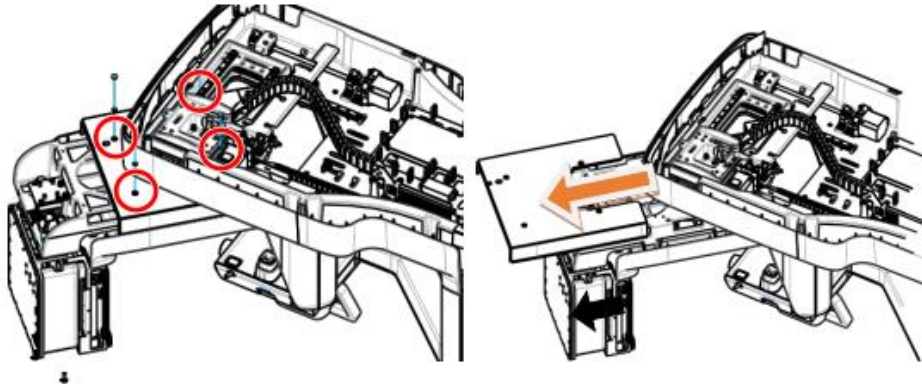

- 2.5 Loosen six screws at the top of the rotator sensor side upper cover to the rotator.

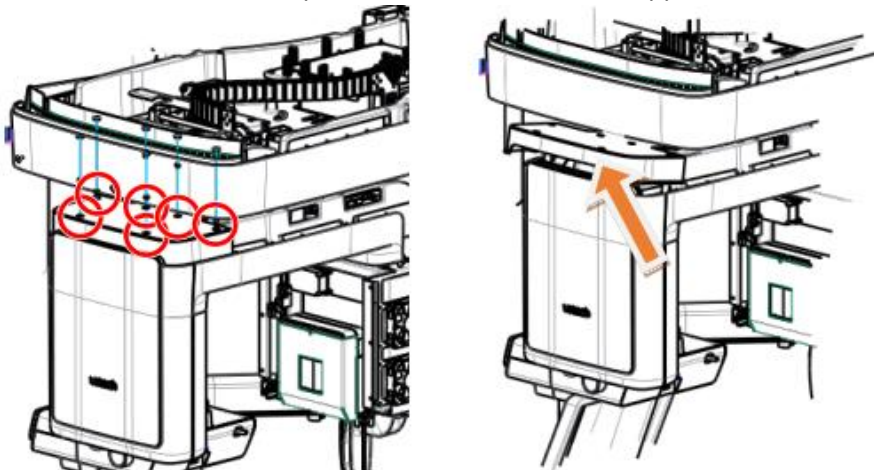


- 2.6 Slide the rotator sensor outer rear cover upward to reveal the screws for the rotator.

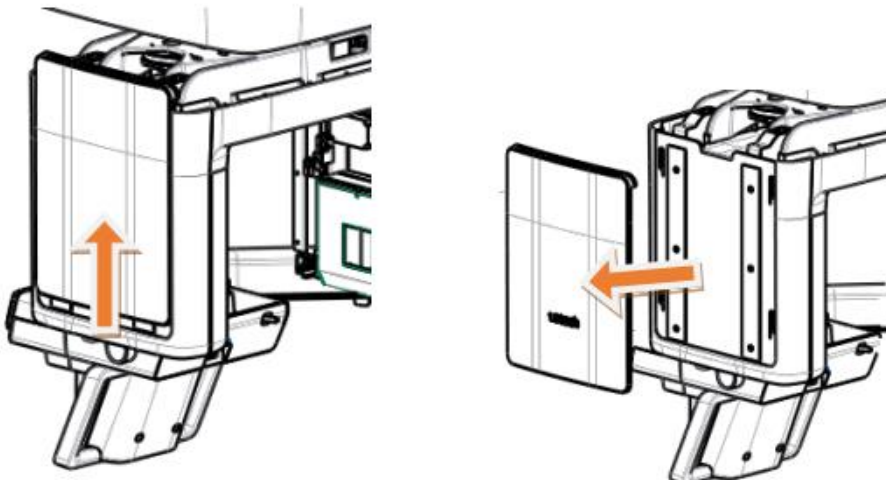


Disassembly of Green X Unit Covers


2.7 Loosen the six screws holding the rotator sensor inner rear cover to remove.

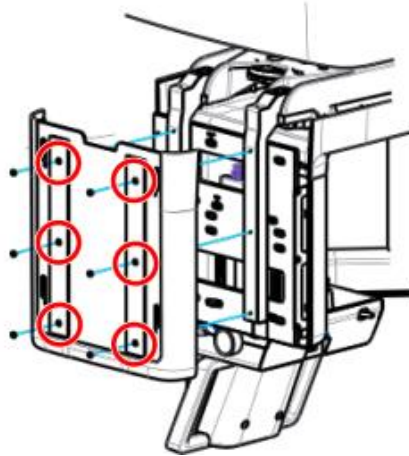

2.8 Loosen the eight screws holding the rotator sensor front cover and slide down to remove.

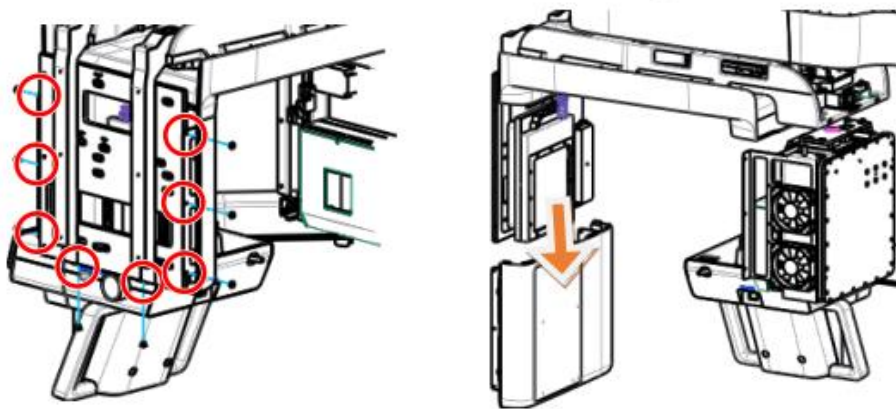


2.9 Loosen six screws holding the rotator bottom cover. To remove the cover, carefully lower the cover from the tube side and swing the cover out away from the unit.

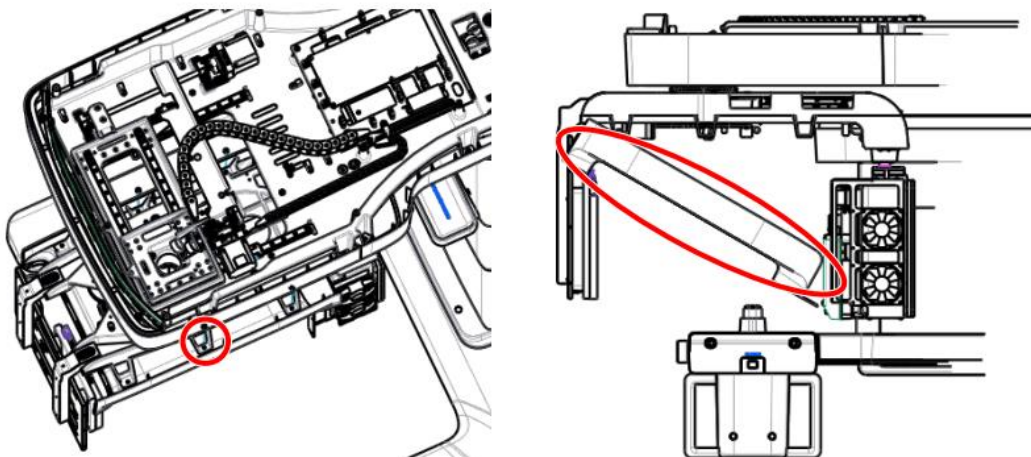


3. DISASSEMBLY OF THE HANDLE FRAME COVERS

3.1 Loosen all screws at the bottom side of handle frame covers.


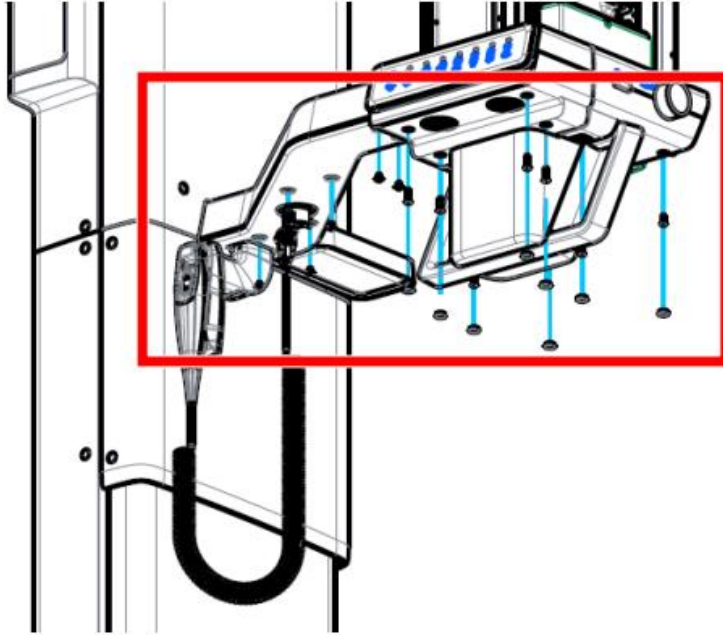


3.2 Lift the top handle frame top cover and chinrest top cover to remove.


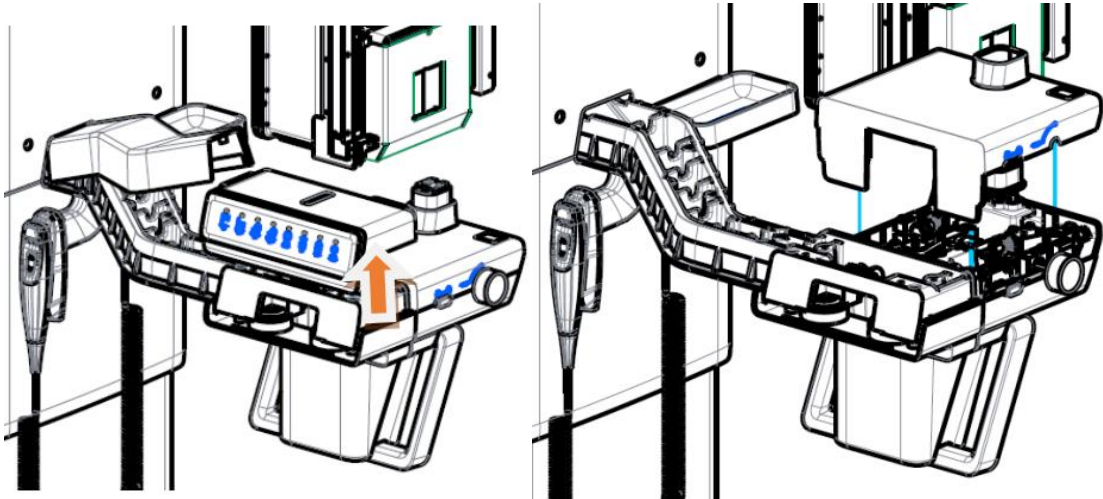


4. DISASSEMBLY OF THE CEPH COVERS

4.1 Loosen six screws at the bottom side of the ceph vertical frame assembly. Lift the top ceph cover to remove.

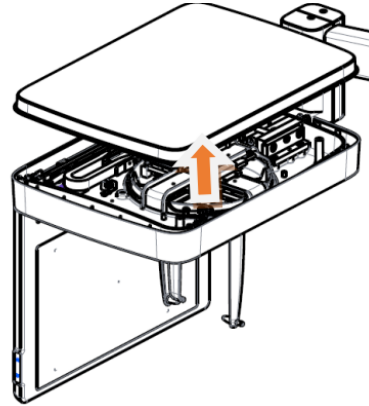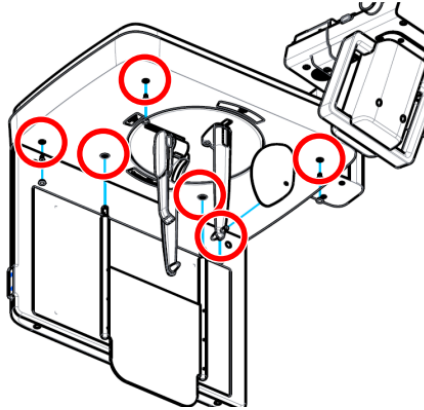

4.2 Loosen all five screws at the top and bottom of the front ceph sensor cover. Pull cover away to remove.

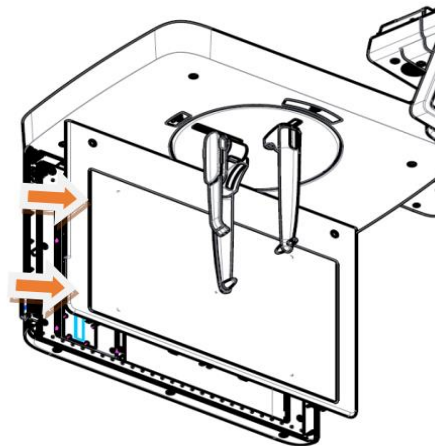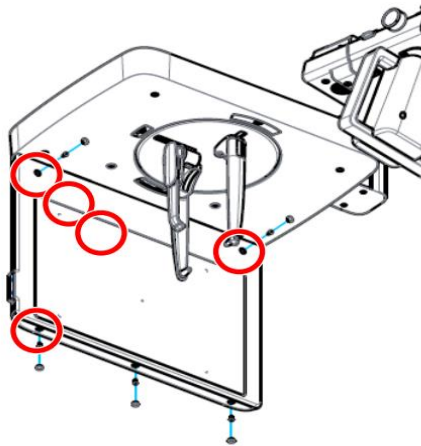

4.3 Loosen all six screw holding the ceph rear sensor cover to the ceph sensor frame. Pull the cover away to remove.

